




SDFI QUICK START GUIDE: BP - G12

Identify the red-labeled contrast camera system (note: it does not have a flash).

Step One – Assemble the Camera


Install Camera Battery

- Locate the **Battery Compartment** located at the bottom of the Camera Body.
- Slide the **Battery Compartment Cover Lock** and Open the cover.
- Insert the Battery until it locks in place.
- Close the **Battery Compartment** by pressing the cover down until it snaps shut.

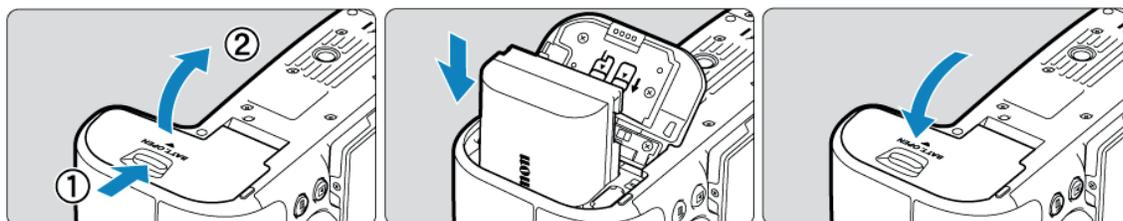

Insert SDFI SD Card

- Slide the **SD Access Cover** to open it.
- Insert the **SDFI SD Card** into **Slot 1**, pressing it gently until it clicks into place. If the **SD Card does not** insert easily, check its orientation and try again.
- Close the Cover and slide it until it **snaps** shut.

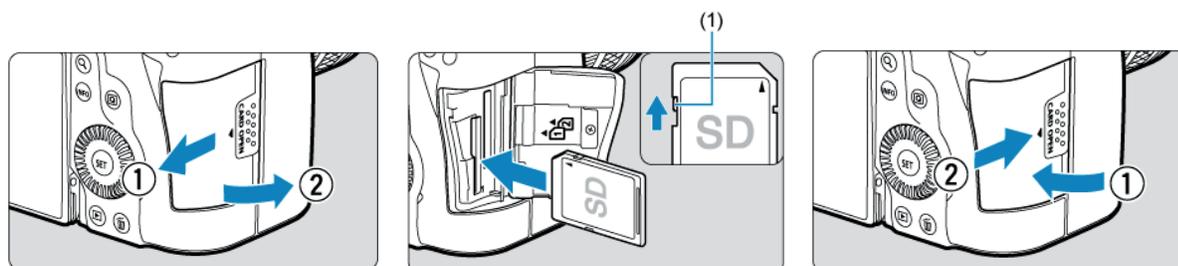




Step Two – Check Camera Settings

Lens Settings

Check that the lens settings are set as follows:

- Focusing Distance Range: **Full**
- Focus Mode: **AF** (Auto Focus)
- Image Stabilizer: **On**

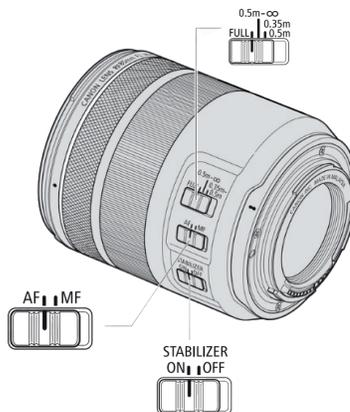


Camera Body Settings


- On the top-left of the camera body, **Set** the camera icon for still photos.



- On the top-right of the camera body, **Set** the **Mode Dial** to "**C1**".

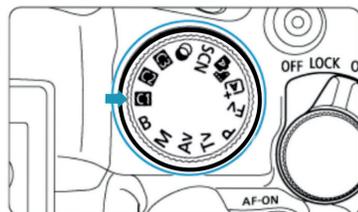

- On the top-right of the camera body, turn the **Power Switch** to **Lock**.

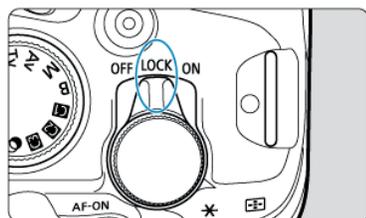




SDFI QUICK START GUIDE: BP - G12

Step Three – Capture an Image

Aperture Settings

- For the **Contrast Camera**, the only aperture setting you'll need to use is **f/5.6**.

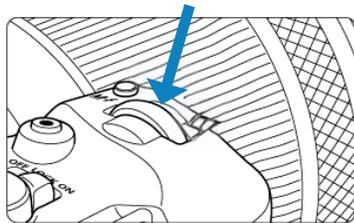

Filter

- Ensure that the **colored lens filter** is attached to the front of the camera lens.

Goggles

- Wear the corresponding colored **goggles**. Both the photographer and the subject must have them on during the exam.

UV Light:

- With your left hand, grip both the **UV light** and the camera body. Keep your right hand on the camera body to press the **shutter button**.
- Position the **UV Lamp** head slightly in front of the camera lens.
- Stay about 8 inches from the area of interest.



Position the AF Frame (Square):

- Check that the white **AF Frame** (Square) is centered on the screen. If it is not centered, press the **SET Button** located to the right of the **screen** inside the **Quick Control Dial**.





Step Three – Capture an Image


Focus on Subject

- Press the **Shutter Button** halfway to activate white **AF Frame** (Square); the white **AF Frame** (Square) will turn green when ready. If the square turns red, step back slightly and try again.



Take the Photo

- Press the **Shutter Button** all the way down and release to capture the photo.

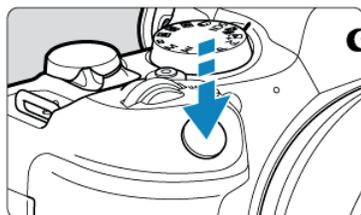





Reviewing Images

Playback Button

- Press the blue **Playback Button**  (located on the lower right backside of the Camera Body, just to the right of the display screen and below the **SET** button).

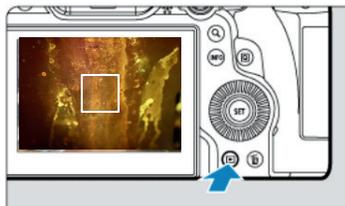


- Browse images by turning the **Quick Control Dial**  or swiping on the **Touchscreen**.

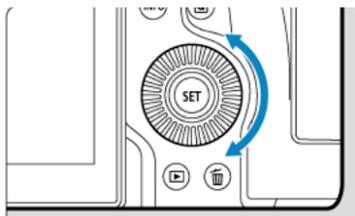


- To magnify an image, touch the **screen** and spread your fingers apart to make the image larger.



- Exit image **Playback Button** by pressing the **Shutter Button** or the **Playback Button** .

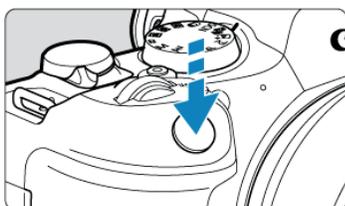


or

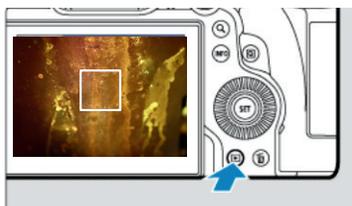



SDFI QUICK START GUIDE: BP - G12

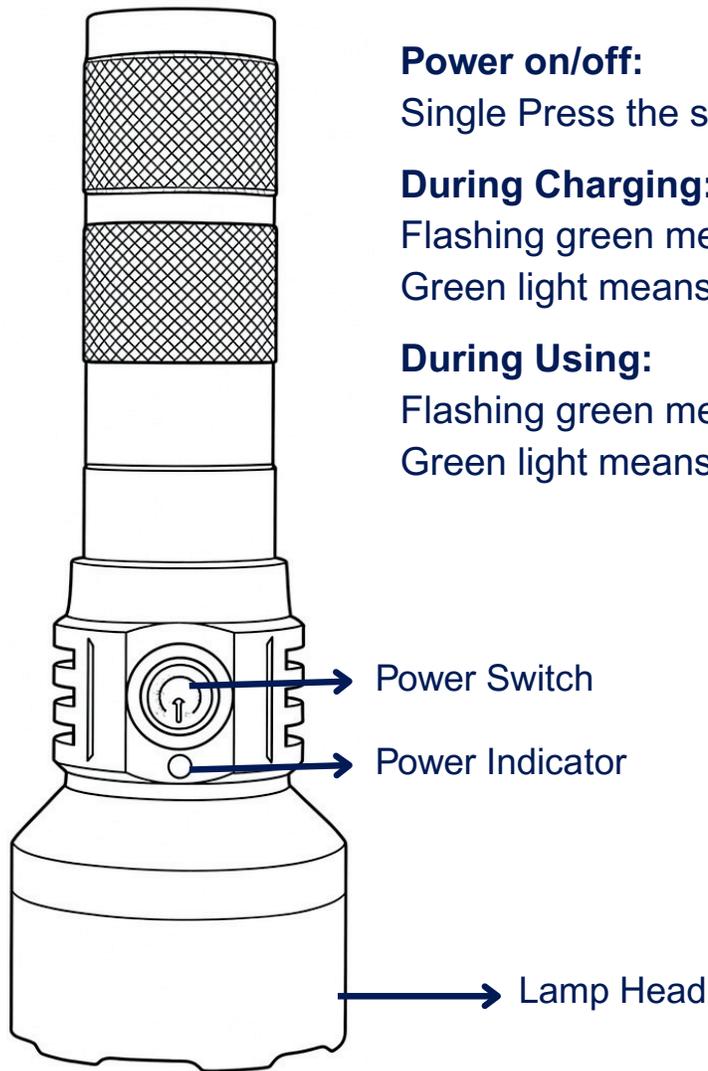

Power on/off:

Single Press the switch button.

During Charging:

Flashing green means charging.

Green light means fully charged.

During Using:

Flashing green means low power.

Green light means high power.

Power Switch

Power Indicator

Lamp Head